





# SARS-CoV-2 ANTIBODY TEST

## COLLOIDAL GOLD IMMUNOCHROMATOGRAPHY



- **ACCURATE**  
High-purity antibody with high accuracy
- **EFFICIENT**  
Test result is available in 15 mins
- **CONVENIENT**  
Suitable for finger blood


- [Sample Type] Whole blood / Plasma / Serum
- [Sample Volume] 20ul / 10ul / 10ul
- [Reaction Time] 15 mins
- [Packing Size] 20 tests per box
- [Storage Condition] 4°C ~ 30°C
- [Shelf Life] 12 months

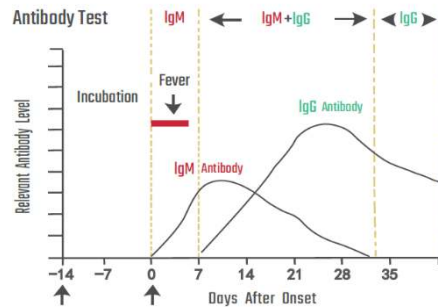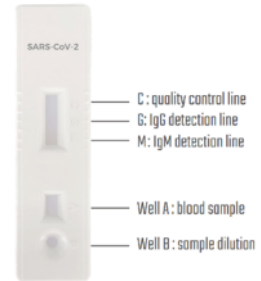


Antibodies will be secreted after virus invasion. Immunoglobulin M (IgM) comes out first, acting as the early sign of infection. Immunoglobulin G (IgG) comes out later, arising a more

### Operation Steps


Step 1 Add 20ul of whole blood sample/ 10ul of serum or plasma sample to well A.

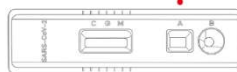

Step 2 Add two drops ( about 80ul ) of sample dilution to well B, and start timing.

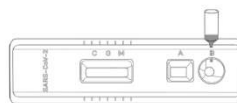

Step 3 The test results should be read within 10-20 mins. Do not read the results after 20 mins.

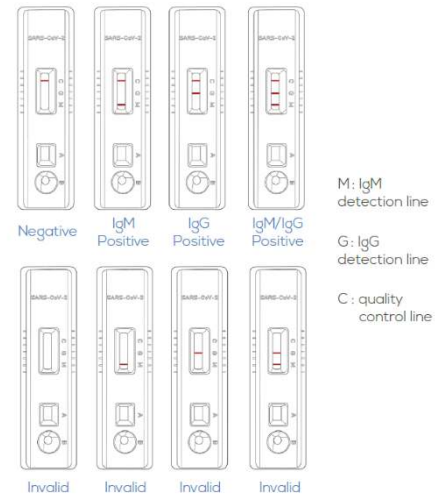

### ADDRESS:

AMBWholesale  
#15 – 270 Esna Park Dr, Markham  
Ontario, L3R 1H3  
Toll-free phone: +1-888-611-5590  
Phone: +1-905-415-0262  
Fax: +1-905-415-0265  
phone: info@ambwholesale.com  
Website: www.ambwholesale.com



Find FDA registration below:

<https://www.accessdata.fda.gov/scripts/cdrh/cfdocs/cfRL/rl.cfm?lid=655739&lpcd=QKO>